
## Avance® Solo

NEGATIVE PRESSURE WOUND THERAPY

## Avance® Solo NPWT System

Manufacturer  
Mölnlycke Health Care AB  
Gamlestadsvägen 3C, Box 13080, SE-402 52 Göteborg, Sweden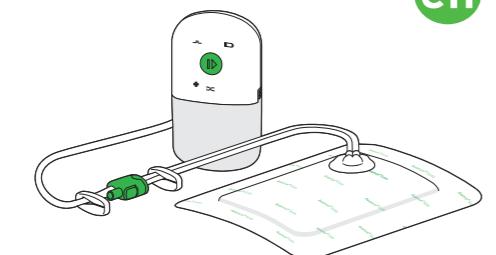

CE 2797

www.molnlycke.com

www.molnlycke.com/symbols

**STERILE EO** The device is sterilized using ethylene oxide

Do not reuse

Caution, see instructions for use

Only use the type and model of lithium batteries specified for this product by Mölnlycke Health Care, see section 13.

Follow instructions for use

Temperature limitation

Keep dry

Keep away from rain

Keep away from sunlight

Keep away from heat

Catalogue number

Use by date / Expiry date

14

Leakage

Blockage

Low battery

Medical Device

Do not use if package is damaged

Manufacturer

MRI unsafe

Humidity limitation

Atmospheric pressure limitation

Applied part type BF

Ingress Protection

ETL Listed Marking

Separate collection of Waste Electrical and Electronic Equipment (WEEE)

System lasts up to 14 days

Leakage

Blockage

Low battery

**1** Introduction

You have received this Patient User Manual because you have been prescribed wound therapy with Avance Solo Negative Pressure Wound Therapy (NPWT) System.

In this Patient User Manual, you will find information and instructions relevant to you as a patient or as a lay caregiver. Read the information carefully and contact a responsible healthcare professional if you are unsure as to the safe use of Avance Solo NPWT System.

Please keep this information in a safe place with your other healthcare documents.

**2** When should Avance Solo NPWT System be used?

Avance Solo NPWT System is indicated for use on patients where application of suction from a negative pressure device would promote wound healing through the removal of exudate and infectious material from the wound. Avance Solo NPWT System can be applied on chronic, acute, traumatic, subacute and dehisced wounds, ulcers (such as diabetic, venous or pressure), surgically closed incisions, flaps, and grafts.

Avance Solo NPWT System is not indicated for use on patients with the following conditions: malignancy in the wound or margins of the wound, untreated and previously confirmed osteomyelitis, non-enteric and unexplored fistulas, necrotic tissue with eschar present, exposed nerves, arteries, veins or organs, exposed anastomotic site.

**3** Warnings

- Therapy with Avance Solo NPWT System shall be applied by a healthcare professional.
- Do not attempt to change the Avance Solo Border Dressing. The dressing shall only be applied, changed and removed by a healthcare professional. If assistance is required, contact your healthcare professional.
- Excessive bleeding is a serious risk with the application of suction to a wound. During therapy, carefully monitor the dressing for excessive bleeding. If sudden or increased bleeding is observed, immediately disconnect the Avance Solo Pump, leave the dressing in place and seek emergency medical attention.
- If you notice a sudden increase in blood pressure or heart rate during therapy with Avance Solo NPWT System, immediately stop the pump and seek emergency medical attention.
- If defibrillation is required, leave the dressing in place and disconnect the pump. Only remove the dressing if the position of the dressing interferes with the defibrillator.
- The Avance Solo Pump should not be used in the presence of or during the following medical therapies:

- Hyperbaric oxygen units
- Environments involving microwaves
- Flammable anesthetics
- Magnetic resonance (MR)
- CT scans and X-rays

The dressing may be left in place unless it is positioned in a location that will interfere with the therapy. The dressing and the foam are safe to use during MR. The impact of the dressing and foam on Magnetic Resonance Imaging (MRI)/Magnetic Resonance Imaging (MRI) imaging artifacts is unknown.


- Make sure to position the pump, canister tubing and dressing tubing so that they do not:
  - cause pressure damage or imprints on the skin of the user
  - trail across a floor where they could be subjected to contamination or present a tripping hazard
  - become twisted or trapped which could block the air path in the tubing
  - become twisted or trapped which could block the air path in the tubing
  - rest on or being exposed to sources of heat
- Regularly check that the negative pressure is active. The pump should indicate normal operation and the dressing should be contracted and firm to the touch.
- If you need to pause the pump, make sure that the dressing is not left without applied suction longer than the time determined by your healthcare professional.
- Products in Avance Solo NPWT System contain small parts which could present a choking hazard. Keep this out of the reach of children.
- Keep products in Avance Solo NPWT System out of the reach of pets.
- If canister or pump is broken, pause the pump and disconnect pump and canister, and contact your healthcare professional.

**4** Precautions

- The Avance Solo Pump has both visual and audible notifications and alarms. Carry or position the pump so that you will be able to detect audible and visual notifications or alarms.
- Regularly monitor the canister mounted on the pump. If the canister appears full or the pump alarms for blockage, change the canister according to instructions provided in this manual, see Section 7.3.
- When the Avance Solo Pump alarms due to a low battery, replace the batteries in the pump according to the instructions provided in this manual, see Section 7.4. Only use the type and model of lithium batteries specified for this product by Mölnlycke Health Care, see Section 13.
- Ensure that the battery lid on the Avance Solo Pump is closed during therapy.
- If you see signs of possible infection such as pain, redness, odor, or sensitization of the wound area or sudden change of wound fluid volume or color, immediately contact a healthcare professional.
- Inform your healthcare professional if you are hypersensitive to the product's materials, see Section 10.
- The Avance Solo Canister is provided sterile. Do not use the canister if the inner packaging of the canister is damaged.
- Do not place the Avance Solo Pump with canister in water or other liquids. If the pump is wet, disconnect the pump and canister and contact your healthcare professional.
- Do not expose the Avance Solo Border Dressing to extensive contact with water. If the dressing is disconnected from the pump, place the dressing tubing so that water does not enter the connector of the dressing tubing.
- Do not take the pump apart.
- Do not modify pump, canister, tubing or dressing as any modification may significantly compromise the ability of Avance Solo NPWT System to deliver therapy.
- The Avance Solo Pump is not intended for use aboard aircraft. During air travel, pause the pump and remove the batteries. Make sure that the dressing is not left without applied suction longer than the time determined by your healthcare professional.

**5** Description of Avance Solo NPWT System

Avance Solo NPWT System consists of Avance Solo Pump, Avance Solo Canister 50ml and Avance Solo Border Dressing with fixation strips for a close seal. Your healthcare professional may have applied Avance Solo Foam under the dressing if you have a cavity wound.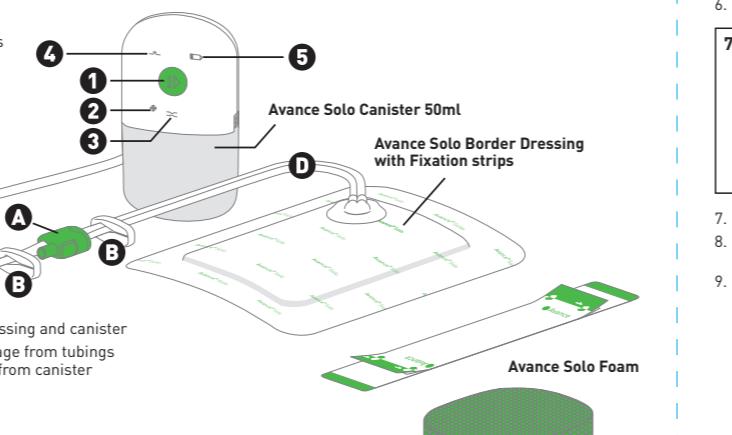

The dressing is applied on the wound and the dressing tubing is then connected to the canister tubing via the green connectors. The canister is attached to the pump. When the pump is started a negative pressure will be created, providing suction to the wound. Fluid from the wound will be transported and absorbed in the dressing. Any excess fluid will be transported from the dressing and collected in the canister. If the canister becomes full, you or your health care specialist will be able to change the canister, see Section 7.3 for instructions.

The pump is operated with a single push button and is battery powered. If the battery power becomes low, you or your health care specialist will be able to change the batteries, see Section 7.4 for instructions. The pump has auditory (beeps) and visual (indicator lights) notifications and alarms to let you know when therapy is being applied as intended or if there is an issue. Make sure that you place the pump so that you can detect the notifications and alarms. Refer to Section 7 and 12 for more instructions and troubleshooting guides.

**6** Daily life while on therapy with Avance Solo NPWT System

**Can you move around while on the therapy?** Based on your health condition, you should be able to move around and maintain your daily activities. Follow the instructions provided by your healthcare professional.

**Will it be painful?** When the dressing is first applied and the pump is started, you may experience a slight pulling or drawing sensation from dressing contraction. If you experience any pain during therapy, please consult your healthcare professional for advice.

**How often will the dressing be changed?** How often your dressing needs to be changed depends on type of wound and how much fluid is collected from the wound. A healthcare professional will determine and give you information on how often the dressing will be changed by a healthcare professional.

**When you are resting or asleep,** place the pump in a safe position where it cannot be pulled off a table or cabinet onto the floor. Make sure that all tubing is placed in a position that minimizes risk of entrapment or strangulation.

**Can I shower?** Light showering is allowed but keep the pump away from water. If the pump is accidentally wet, disconnect the pump and canister and contact your healthcare professional. The dressing is water resistant but should not be exposed to jets of water. For light showering; pause therapy by pressing and holding the green push button on the pump, and release after two (2) seconds. Clamp the canister tubing and dressing tubing by positioning the slide clamps next to the green connectors and slide across the tubings until secured. Disconnect the canister tubing from the dressing tubing. Ensure that the dressing tubing is not placed into contact with water.

**Can I clean the pump?** You can clean the pump by wiping with a damp cloth or with a non-abrasive detergent. Do not put the pump under running water.

**7** Handling Instructions**7.1 How do I know that Avance Solo NPWT System gives therapy as intended?**

When Avance Solo NPWT System gives therapy as intended, the green push button on the pump will flash 2 times every minute and the dressing will have a wrinkled appearance and be firm to the touch. Regularly check that negative pressure is active by monitoring visual and audible notifications and alarms from the pump.

**NOTE:** When the pump is first started, the green push button on the pump flashes once every second for 15 minutes.

**7.2 How do I know that the pump is paused?**

When the pump is paused you will hear an audible notification as two short beeps and the green push button will stop flashing, all indicator lights will be off.

The pump will repeat the two short beeps every 15 minutes as long as the pump is paused.

**NOTE:** If you do not restart the pump manually, it will automatically restart after 60 minutes.

**7.3 How do I know when to change the canister?**

The canister should be changed when you can see that it is full through the transparent window on the back of the canister. A Blockage alarm from the pump may also indicate that you need to change the canister. The BLOCKAGE indicator light flashes once every second, the pump repeatedly alarms with a beep and will then pause.

**NOTE:** A Blockage alarm may also be triggered by blockage in the tubings. Always make sure that the tubings are not clamped or kinked.

**7.4 How do I know when to change batteries?**

The pump will notify you when the battery power is reaching its end, the pump will indicate the following:

All indicator lights and the green push button on the pump will flash with high intensity and the pump will notify with three beeps at different frequency tones: one beep with high, one beep with medium followed by one beep with low frequency tone.

**7.5 How do I know that the therapy time of 14 days is complete?**

The Avance Solo Pump is for single patient use, is battery powered and has a 14-day lifespan.

When the therapy time of 14 days is reached, the pump will indicate the following:

All indicator lights and the green push button on the pump will flash with high intensity and the pump will notify with three beeps at different frequency tones: one beep with high, one beep with medium followed by one beep with low frequency tone.

**7.6 What do I do if the green push button is accidentally pressed?**

When the green push button is accidentally pressed, the pump will notify with a beep.

No action is required.

**8** Disposal

When you have changed the batteries, dispose the batteries so that they can be recycled in accordance with requirements by local regulations, relevant state laws and the Waste Electrical and Electronic Equipment Directive (WEEE).

When you have changed the canister, make sure that you dispose it as clinical waste in accordance with local regulations.

Ask your healthcare professional for more information if you are unsure as to the safe disposal. You can also find more information about safe disposal at [www.molnlycke.com/wastehandling](http://www.molnlycke.com/wastehandling).

**9** Caution

Avance Solo NPWT System must be used in accordance with instructions provided in this Patient User Manual. Read these instructions before using the system and have them available during use. Failure to read and understand these instructions may lead to misuse of the system and improper performance. These instructions are a general guide for the use of the product. Specific medical situations must be addressed by a healthcare professional.

**10** Material content

Dressing, fixation strips: polyethylene, polyurethane, polyester, super-absorbent particles, viscose fiber, soft silicone, polycrylic adhesive


Foam: polyurethane

Canister: polycarbonate, polyurethane

Pump: polycarbonate, acrylonitrile butadiene styrene, thermoplastic elastomer

Tubings with clamps: polyolefin based thermoplastic elastomer, polyethylene

Connectors: acrylonitrile butadiene styrene, thermoplastic olefin, polyethylene

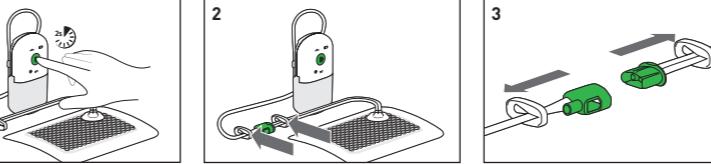**To replace the canister, perform the following steps**

- If the pump is active, pause the pump by pressing down the green push button, and release after two (2) seconds.
- Clamp the canister tubing and dressing tubing by positioning the slide clamps next to the green connectors and slide across the tubings until secured. Blocking the tubings minimizes fluid leakage when you disconnect the canister from the dressing.
- Disconnect the canister tubing from the dressing tubing by squeezing the connector from both sides and pull apart.
- Remove the canister by pushing the spring buttons on both sides and pull.
- Attach a new canister to the pump by pushing the canister until it clicks on both sides.
- Be sure to attach the canister tubing onto the back of the pump.

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